


ASSEMBLY INSTRUCTIONS: ARM REST

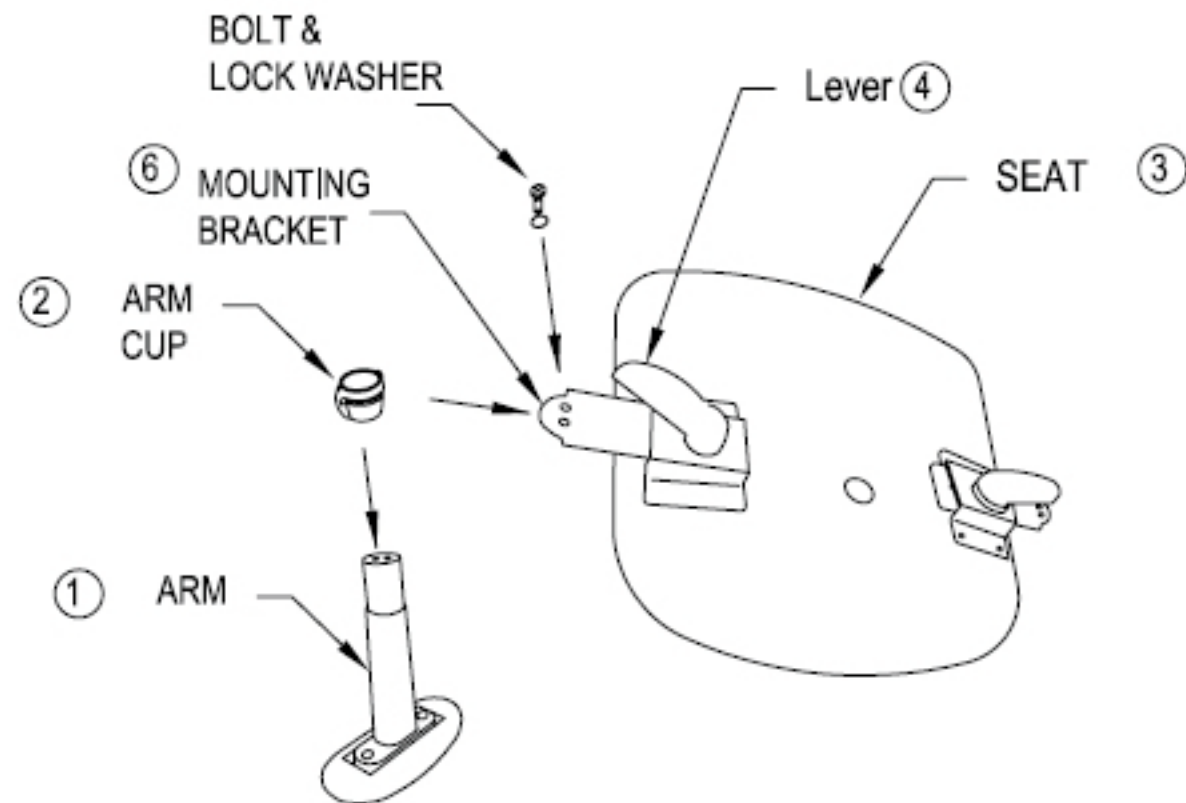

- 1) Remove the two bolts and lock washers from the Arm (1).
- 2) Align the open slot of the arm cup (2) so that it is even with the bottom of the cylinder of arm (1) shown in (figure 1).
- 3) With the seat (3) upside down, align the slot of the arm cup (2) with the mounting bracket (6). Attach with the two bolts and lock washers. (Note: make sure that the button (5) is facing away from the seat (3)). Repeat steps for opposite side.
- 4) You can adjust the arms in and out by loosening lever (4) and sliding the arm in or out. When adjusted to desired location, tighten lever (4).
- 5) You can adjust the arms up or down by holding in button (5) adjust to desired height and then release button (5).

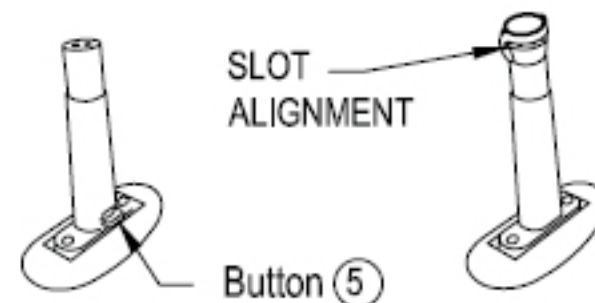


FIGURE 1